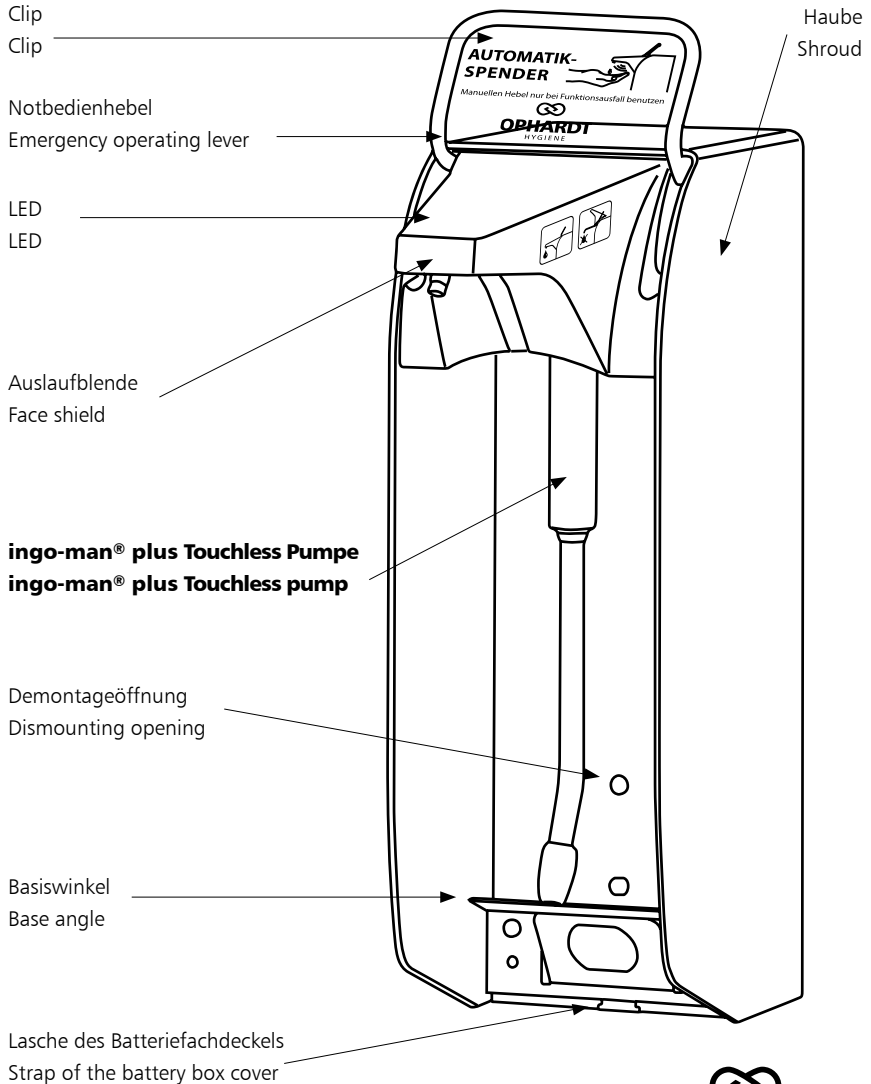
ingo-man® plus Touchless

ingo-man® plus Touchless XT

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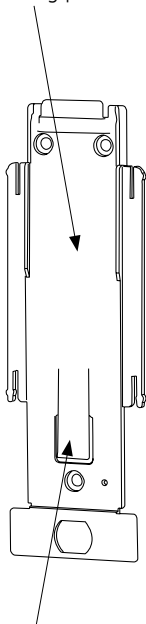
Please read through carefully these operating instructions prior to initial operation and keep it.

In case of a resale of the apparatus, please pass on these operating instructions to the buyer.

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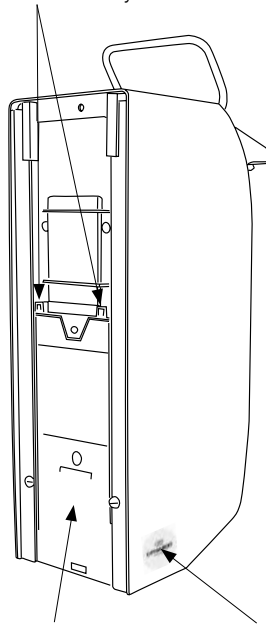
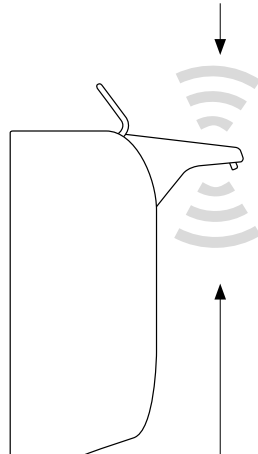
1.	Notes for the user of the operating instructions	27
2.	Safety instructions	27
2.1	<i>General safety</i>	28
2.2	<i>Safety during operation</i>	28
3.	Use according to regulations	29
3.1	<i>Orderly handling of the dispenser</i>	29
4.	Component parts of the dispenser/scope of delivery	30
5.	Technical description	31
6.	Dispenser mounting	32
6.1	<i>Preparation for installation</i>	33
6.2	<i>Mounting</i>	34
7.	Going into operation	35
7.1	<i>Insert of the batteries</i>	35
7.2	<i>Use of the appliance</i>	37
8.	Basic operation	38
9.	Change of the pump	39
9.1	<i>Exchange of the pump</i>	39
9.2	<i>Changing the dosage</i>	40
10.	Exchange of the receptacle	40
11.	Operation with emergency operating lever	41
12.	LED display	41
13.	Insert of the locking plate	42
14.	Maintenance / attendance and maintenance	43
14.1	<i>Cleaning and attendance</i>	43
14.2	<i>Battery change</i>	44
14.3	<i>In time refill of the medium</i>	44
15.	Risks and Troubleshooting	45
15.1	<i>Risks</i>	45
15.2	<i>Troubleshooting</i>	46
16.	Disposal	47
17.	Technical data	48
18.	IMP Touchless mains operated	49

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1. Indications for the use of the service instructions

The following symbols are used in the instructions

Safety reference:

A non-observance may cause damage to persons or defects at the device

Notice:

Supplementary information concerning the operation of the appliance

2. Safety information

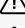
Attention!

OPHARDT does not assume any liability for damages in the case of:

- defects at the appliance caused by mechanical influences or overvoltage
- changings at the appliance without express approval of **OPHARDT**
- use for other purposes than that mentioned in the application

2. Safety information

2.1 General safety

-  • Repairs at the appliance may exclusively be carried out by **OPHARDT** employees or persons authorized by **OPHARDT**. Important dangers may occur by unexpert repairs. In case of a necessary repair please contact the **OPHARDT** service department.
-  • Suspend the appliance on a dry and splash-proof place.
-  • This appliance is not a toy. Keep and use it out of the reach of children.
-  • The appliance is not suitable for the transportation of media.
-  • The appliance is not suitable for the metering of other than the indicated media.
-  • Exclusively operate the appliance with leakproof alcalic batteries of the same manufacturer and type and with the same charging condition. Only use batteries with the indicated size type D.
-  • Do not operate the appliance with rechargeable batteries (accumulators).
-  • Do no longer use the apparatus if batteries have leaked in the appliance.
-  • Never use force.
-  • Take the batteries from the appliance prior to cleaning and attendance.
-  • Dispose of the empty batteries at the collecting points.
-  • The appliance needs not be disassembled, no repairable parts.
-  • The face shield is not removable.

2.2 Safety during operation of the appliance

-  • Protect your appliance against humidity.
-  • Do not expose the appliance to thermal radiation and heat.
-  • Never submerge the appliance in the water.
-  • Only fill the appliance with the indicated substances in accordance with the intended use.
-  • There are no dismountable parts with the exception of pump, receptacle, battery cover, mounting plate, batteries, sealing diaphragm (accessories)
-  • The appliance is not autoclavable and cannot be cleaned in the dishwasher.
-  • The apparatus shouldn't be cleaned with water jets or by means of a high-pressure cleaner.
-  • Please make sure that the medium does not reach the ground.

EN 3. Intended use

3. Intended use

The appliance is suitable for the touchless hands' disinfection. Furthermore the appliance can be used for the dispensing of liquid soaps for washing one's hands. It is necessary to rub the medium into the hands.

The apparatus is exclusively designed for battery operation.

Always hang the dispenser upright on to the wall. The dispenser shouldn't be mounted freely. The appliance can only be used in closed rooms.

Use the appliance for hands' disinfection and cleaning with the usual application cycles.

Ambient conditions:

- ⚠ • Never use the appliance in potentially explosive areas.
- ⚠ • Only use the appliance in a dustless environment.
- ⚠ • Pay attention to that the apparatus is only used at a relative air moisture of <90% (not condensing).
- ⚠ • Only use this apparatus at room temperature.

Media to be used:


- ⚠ • The used medium has to be free-flowing at room temperature.
In case of doubt please test your medium for suitability.
- ⚠ • The appliance is designed for operation with disinfectants or with liquid soap.
- ⚠ • The appliance is not suitable for operation with abrasive and aggressive media.
- ⚠ • Foodstuff as medium is not permissible.

3.1 Operation of the dispenser according to the regulations

In order to always guarantee an optimum functioning of the dispenser the following has to be observed:

- Use of the dispenser only with original parts.
- Only use disinfectants or thin liquid soap.
- When inserting the batteries please absolutely pay attention to the correct polarity!
(Pay attention to the representation in the case).
- The dispenser may not be located close to radiation-emitting or statically charged objects while it is used.
- The dispenser only meters with closed face shield.
- In general (and especially during operation of the dispenser) a contact with the face shield has to be avoided.
- The emergency operating lever has to be actuated exclusively in case of a breakdown of automatic function!

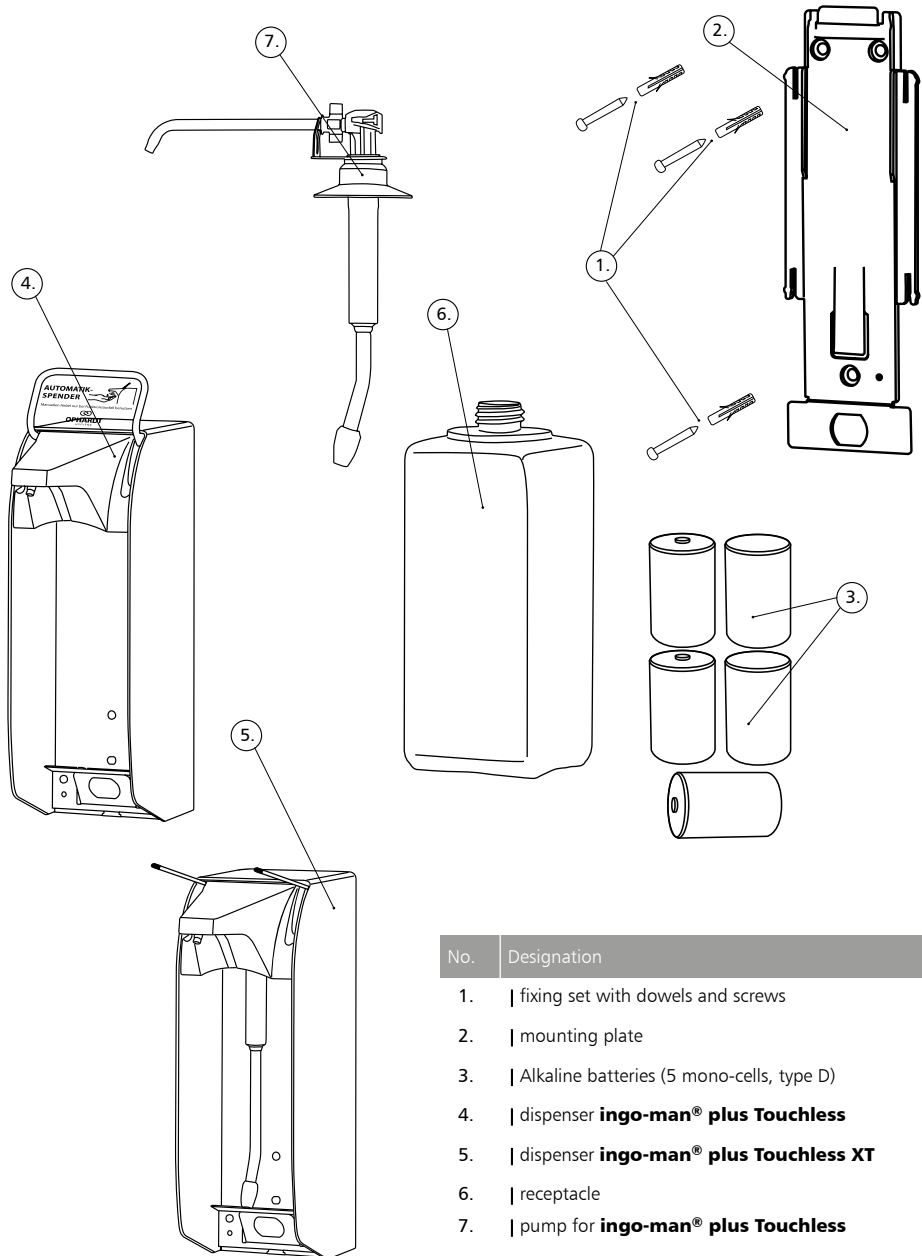EN 4. Component parts of the dispenser / scope of delivery

No.	Designation
-----	-------------

- | | |
|----|--|
| 1. | fixing set with dowels and screws |
| 2. | mounting plate |
| 3. | Alkaline batteries (5 mono-cells, type D) |
| 4. | dispenser ingo-man® plus Touchless |
| 5. | dispenser ingo-man® plus Touchless XT |
| 6. | receptacle |
| 7. | pump for ingo-man® plus Touchless |

EN 5. Technical description

5. Technical description

The appliance meters free-flowing media by means of a removable suction pump from an exchangeable receptacle which is inserted into the dispenser. The dosing quantity per pump stroke can be modified by adjusting it on the pump with a suitable tool (e.g. flat pliers).

The appliance recognizes by means of a touchless sensor technique and a hand approaching the outlet and, thus, automatically releases a pump stroke. Further pump strokes are carried out if the lower hand remains uninterruptably in the lower sensor area and a second hand is lead to the upper sensor area.

EN 6. Mounting of the dispenser

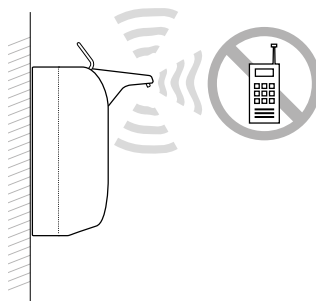
6. Mounting of the dispenser

6.1 Preparation of the installation

Place the dispenser above the washstand or right next to it. If this is not possible, one has to take care by means of adequate equipment that no medium pours out on the floor.

• Prior to the use of new filling media tests are required!

• Please pay attention to that during installation no radiation-emittent object (i.e. X-ray apparatus, mobile telephone... etc.) is in the near proximity. Electromagnetic interference sources and/or radiating electric fields may impair the function.


Prior to the mounting of the dispenser please observe the following

• **Required place above the dispenser:**


40 mm (for putting on and removing from the mounting plate).
Please take the required place below the dispenser from the dimensioned sketch.

• **Suitable mounting height:**

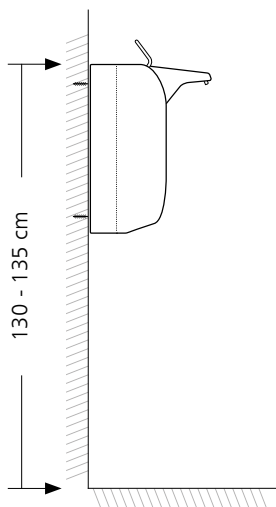130-135 cm, measured from the floor up to the upper edge of the dispenser.


EN 6. Mounting of the dispenser

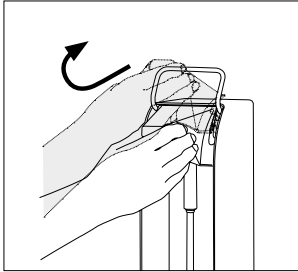
6.2 Mounting

For the mounting and going into operation of the **ingo-man® Touchless** please keep the following steps in sequence.

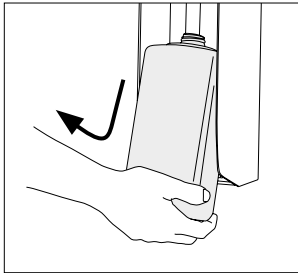
Open face shield. Therefore, pull it slightly to front and guide it upwards.

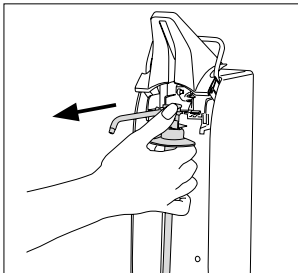When delivered the mounting plate is in the dispenser. For the removal of the mounting plate slowly lift the receptacle in the dispenser and remove it from the dispenser.

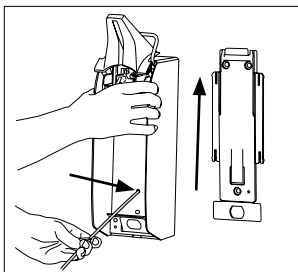
Remove the pump (see " pump change under point 9"), with opened face shield.


Insert solid, thin object (for instance a screw driver) through the dismantling opening in the front side of the dispenser and press down the mounting tongue of the mounting plate. With pressed down mounting tongue pull off the dispenser from the mounting plate.

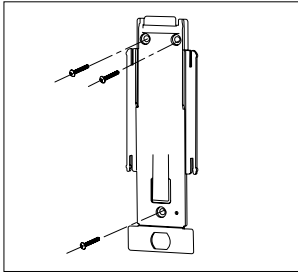EN 6. Mounting of the dispenser


Fix mounting plate at the wall in desired position by means of dowels and screws.

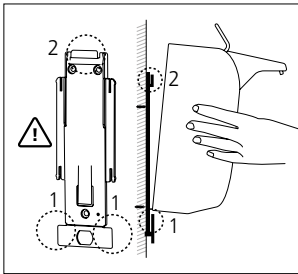
Drilling diagram see technical data (see point 17).

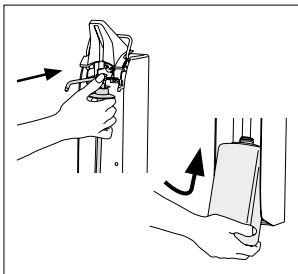Slide the dispenser from above on to the mounting plate until it clicks into place.

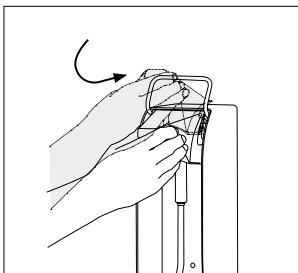Reinsert first the pump and then the receptacle.

Close the face shield.


EN 7. Going into operation

i • The batteries should be replaced in the same type as the battery supplied. We recommend the use of zinc manganese batteries, also known as alkaline manganese batteries, to ensure a long service life.


7. Going into operation

7.1 Insert the batteries

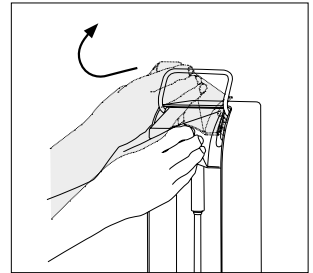
The dispenser has to be removed from the mounting plate (see removal of the mounting plate 6.2), the back of the dispenser must be easily accessible.

Open face shield.

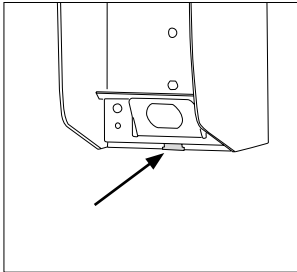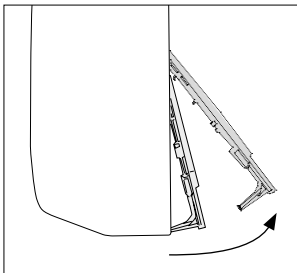
Open the cover of the battery case. To do that release the lower strap of the cover which is clipped in below the base angle. Then unhinge the cover and put it aside.

Now insert the batteries into the dispenser. Please pay attention to the correct polarity.

Keep the sequence as follows!

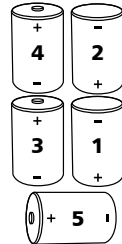Please remove batteries in the same numbered sequence. *No. 5 last.


i • If the dispenser is not used for a longer period than 3 months, the batteries should be removed in order to prevent unwanted discharge.

EN 7. Going into operation

- After correct insertion of the batteries an acoustic signal sounds.
- Slip the dispenser from above onto the mounting plate.
- Close the face shield:
now the dispenser starts with the first gauging.
- Noise level of the unit is below 70dB(A)

i • If the batteries have not been inserted with closed face shield, the measuring procedure has to be carried out once again after having put the dispenser on the mounting plate by shortly opening and closing the runout diaphragm such as with the change and the insertion of the pump. This procedure will be explained below in more detail.

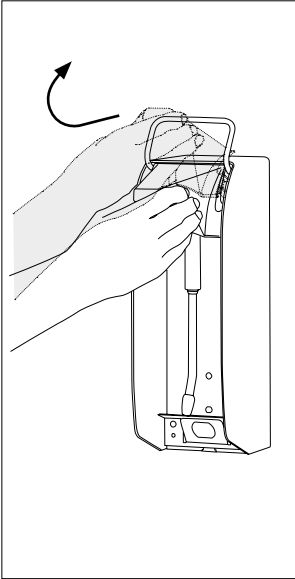EN 7. Going into operation

7.2 Use of the appliance

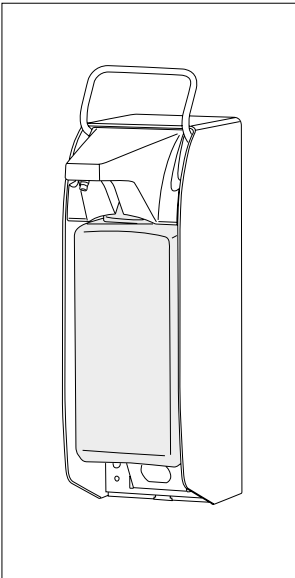
- With closed face shield it may happen during the change of the receptacle, that the lower sensor is actuated unintentionally when inserting the receptacle and medium penetrates. This can be prevented if the face shield remains opened during the change/insertion of the receptacle.

Whenever the face shield is closed, the dispenser adjusts to the pump and then to surrounding area. Therefore all parts of the body as well as all objects have to be removed rapidly from the immediate vicinity after having closed the face shield.

After the change of the receptacle and/or the pump the medium has to be pumped by repeated action (strokes).


During that hold the hand under the outlet.

The LED lights while the dispenser gauges. As soon as the light is out the dispenser is ready for operation.

- With the correct opening and closing of the face shield a different acoustic signal sounds each time.
- After the sliding on to the mounting plate the dispenser has to be adjusted to the pump and its environment – this happens automatically after each closing of the face shield at the end of the gauging procedure(as described above).
- If necessary, insert the pump.
- Now insert a receptacle with the suitable medium of your choice into the dispenser. Insert it from below via the pump and put the receptacle with the back edge of the bottom of the receptacle onto the base angle.
- ⚠ • Please make sure that the receptacle is in the right position and can't slip down out of the dispenser.
- Now you can use the appliance.

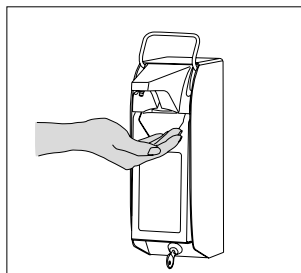
EN 8. Basic operation

8. Basic operation

The actuation of the dispenser is effected if a hand approaches from below under the outlet. At a distance of approx. 4 cm the dispenser releases automatically a pumping process.

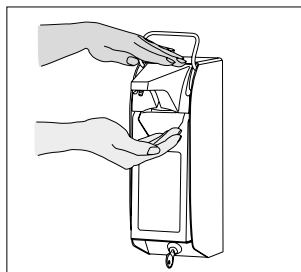
For this form a hollow with your hand.

For further dispensing actions you can hold your lower hand within the lower sensor-controlled area and approach the second hand from above to the face shield ("virtual operating lever").

When actuating the dispenser with the second hand, please pay attention to that one hand remains under the outlet.

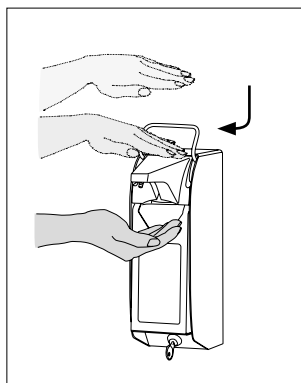i • If you approach with only one hand from above without holding the other hand below the outlet, the appliance does not pump.

Approach the upper hand, to up to a distance of approx. 4 cm to the face shield – the dispenser actuates several times.


The dispenser meters as long as the upper hand is not removed - up to 5 times.

You may also approach your hand from above several times in sequence as if you actuated a virtual operating lever.

The pumping action is interrupted when removing both hands.

The complete action can be repeated as many times as you like.

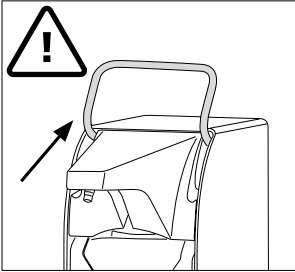
EN 9. Pump change


9. Pump change

9.1 Exchange of the pump

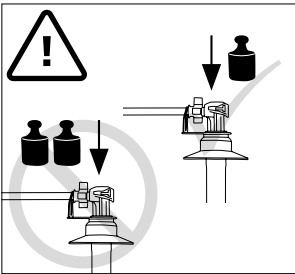- ⚠ • The pump can only be changed if the emergency operating lever is in upper position!
Otherwise serious damages on the appliance may occur.

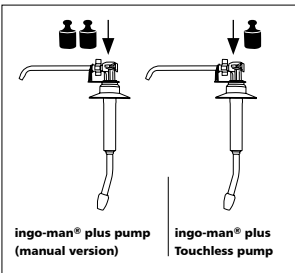
- ⚠ • Please take care that only original **OPHARDT Touchless-pumps** will be used. Do not use a pump of the type **OPHARDT ingo-man® plus** (with strong spring), because this might lead to damages at the dispenser.


You can recognize the **ingo-man® plus pump (manual version)** by the fact that in contrast to the **ingo-man® plus Touchless pump** it develops a manually perceptible, relatively strong counter-pressure when pressing it down with the finger.

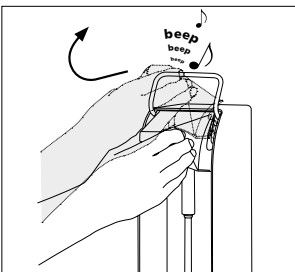The **ingo-man® plus Touchless pump**, however, can be pressed down very easily.


Swivel the face shield upwards (see 7.2).
After the opening an acoustic signal sounds. When opening the face shield with the hand it is recommended to approach first with the hand from above (activation of the upper sensor area) in order to avoid unintentioned pumping actions.

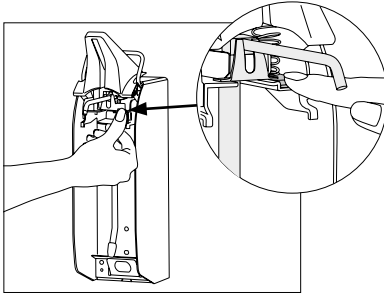
EN 9. Pump change

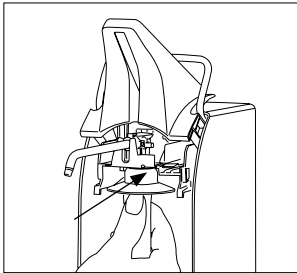Push and hold the pump catch (key „PUSH“) with the finger and pull out the pump to the front.

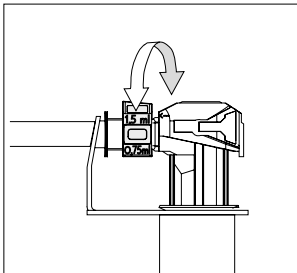
Push in the pump from the front side into the intended guidance until it snaps in. Reclose the face shield.

9.2 Change of the dosage

The dosage of the pump is adjusted to the maximum of approx. 1.5 ml/stroke. If required the dosage can be reduced in stages by turning the dosage piece (part of the pump) up to approx. 0,75 ml/stroke. With first use, the wheel may be hard to turn; if necessary use a pair of flat-nose pliers.

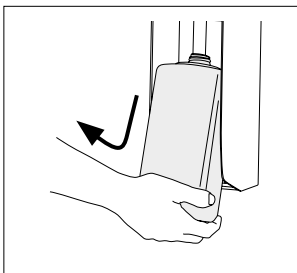For this remove the pump from the dispenser.

10. Exchange of receptacle

- Lift up the empty receptacle from below, draw it to the front and remove it from the dispenser.
- Insert the filled-up receptacle from below into the dispenser. Please pay attention to the correct position of the receptacle (see 7.2).
- If necessary, remove the locking plate before.


i • In order to avoid unintentioned pumping actions, take care that the face shield is opened.

EN 11. Operation with emergency operating lever


11. Operation with emergency operating lever

In case of a failure of the automatic functions (for instance batteries low), an operation by means of the emergency operating lever is possible.

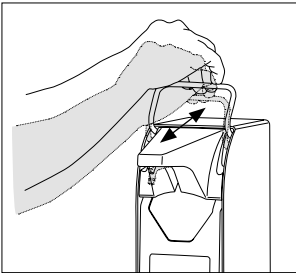⚠ • Only actuate the emergency operating lever in case of a failure of the automatic function !



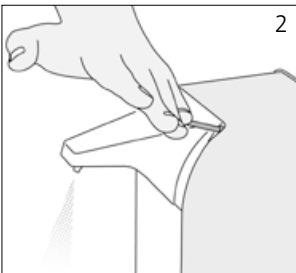
For this purpose press down the lever to the lower limit stop and return it again manually back up to the initial position.

Design ingo-man® plus Touchless XT

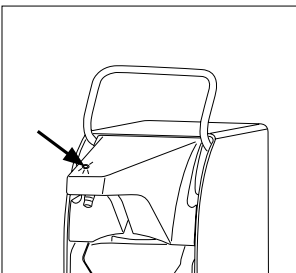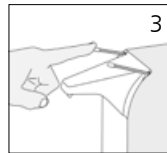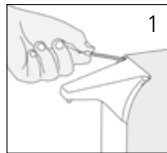By pulling the emergency operating levers gently forwards (fig. 1) and pushing them down the dispenser can be used manually (fig. 2). Push all parts back into the original position (fig. 3).


12. LED display

For protection against unintentioned actuation, electromagnetic interference sources and false handling are recognized and indicated by the lighting up of the LED. As long as the LED lights up, the automatic pumping action is suppressed.


i • The LED also lights up during gauging. When it expires, the dispenser is ready for operation.

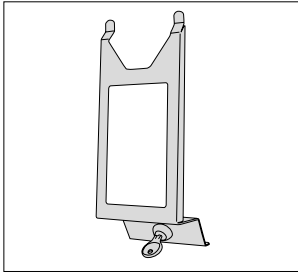EN 13. Insertion of the locking plate


13. Insertion of the locking plate

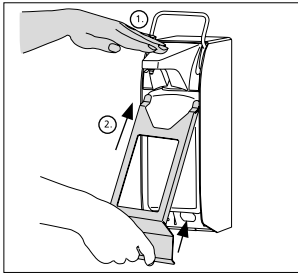The locking plate is an optional accessory which prevents an unauthorized access to the medium.

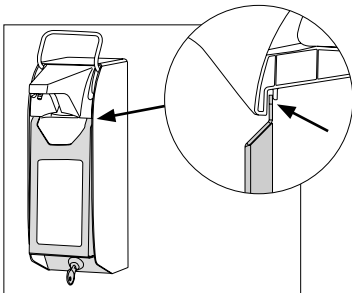- When inserting the locking plate the face shield has to be closed as it is locked, too.
- At first, hold one hand underneath the face shield in order to block unwanted pumping actions.
- Insert the sealing locking plate from below into the face shield.

- Pay attention to the correct position in the face shield.
- Then insert the lock into the intended sparing (press the locking plate in the lower area towards the back) and lock up.
- For the removal proceed conversely.

i • Note:

During the insertion of the faceshield it can happen due to the handling that the acoustic battery level indication will be activated (see 14.2). Please don't be irritated in case that you hear several acoustic signals.

EN 14. Maintenance / attendance and maintenance**14. Maintenance / attendance and maintenance****14.1 Cleaning and attendance**

Please consider that also a high-quality product like this **ingo-man®** appliance requires attendance and care. The dispenser itself is maintenance-free with the exception of the battery exchange!

The Robert Koch Institute recommends: *“The scope and frequency of checks and the reprocessing of dispensers should be specified in the internal hygiene plan, e.g. every six months.”*
[Source: Robert Koch Institute, Recommendation of the Commission for Hospital Hygiene and Infection Prevention 2016]

For cleaning the dispenser remove the mounting plate and dismount the batteries as it is described under 6.2. Cleaning of the dispenser only from the outside. Wipe the apparatus with a humid cloth.

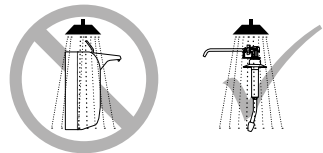
Please only use warm water and do not use caustic (chlorous) purifiers or scouring powders!

- ⚠ • In order to avoid operating errors during cleaning, we recommend to remove the receptacle and the pump out of the dispenser and to block the pumping function by actuating the upper sensor.

If the receptacle is filled up, a non-use during a longer period of time does not lead to an impairment. For that reason exchange the empty receptacle in time. If the suction valve is nevertheless clogged, please only pump hot water through it in order to restore the function.

- ⚠ • **By no means disassemble the appliance!**

- ⚠ • **Please pay attention to that the appliance is not submerged or exposed to water jets.**

- ⚠ • **The appliance is not autoclavable and cannot be cleaned in the dishwasher.**


- ⚠ • **The pump itself is dishwasher-proof and autoclavable, however it must rest for about 6 hours after such a treatment before it is inserted again into the appliance.**

EN 14. Maintenance / attendance and maintenance

14.2 Exchange of battery

Test the battery level:

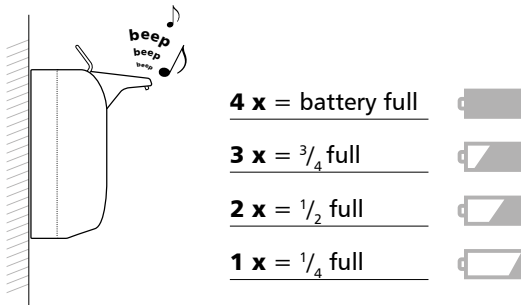First of all approach one hand to the upper sensor and then the other hand to the lower sensor. Remain in this attitude for at least 3 seconds until the acoustic signal sounds:

- In case of weak batteries during normal use a short acoustic signal sounds instead of the pumping action to indicate that the battery has to be replaced.

14.3 In time refill of the medium

For an interruption-free hygienic supply the receptacle has to be exchanged in time.

15.1 Risks and Troubleshooting

15.1 Risks

In dealing with the **ingo-man® plus Touchless** please take notice of some important differences compared to our manually operated dispensers of the **ingo-man®** product range, especially the **ingo-man® plus (manual version)**:

- The **ingo-man® plus Touchless** is not autoclavable
- The pump of the **ingo-man® plus** can't be inserted into the **ingo-man® plus Touchless**, because this can lead to failure and defects on the appliance.
- The discharge diaphragm of the **ingo-man® plus Touchless** is not removable.
- The sealing diaphragm of the **ingo-man® plus** is not compatible with the **ingo-man® plus Touchless** and vice versa!

Other risks:

- Never try to remove or to change the pump when the emergency operating lever is not locked in the upper position!
- The emergency operating lever has to be actuated only in case of a failure of the automatic function!
- Do not carry out any repair at the appliance!


EN 15.2 Troubleshooting

Characteristic	possible cause	solution
1. After insertion of the batteries no acoustic signal sounds	<ul style="list-style-type: none"> Batteries are low. Batteries have not been inserted correctly. Face shield closed. 	<ul style="list-style-type: none"> Insert new batteries. Open the battery box, insert batteries according to the diagram. Pay attention to the polarity and the sequence. Remove batteries, open face shield, insert batteries again.
2. During opening and closing of the face shield no signal sounds	<ul style="list-style-type: none"> Batteries are low. No batteries inserted. Battery contact interrupted. Appliance defective. 	<ul style="list-style-type: none"> Insert new batteries. Insert batteries. Restore battery contact. Call after-sales service.
3. The LED lights up if a hand is hold below the dispenser but no medium penetrates	<ul style="list-style-type: none"> Operating error. Appliance recognized a disturbance (unwanted emission or electric field). 	<ul style="list-style-type: none"> Operate appliance according to point 8. Pay attention to the sequence of actuation in the sensor areas. Avoid contact with the face shield. A) Remove all electromagnetically charged or emitting appliances out of the reach of the sensor. B) Open and close the face shield and quickly remove hands and/or other disturbing objects. Wait for a short time until the dispenser has gauged. Then actuate the dispenser as usual. C) Remove receptacle and pump and reinsert them. D) Make sure that the pump has the right position.
4. The appliance seems to function normally but no medium penetrates.	<ul style="list-style-type: none"> No receptacle is inserted. No pump is installed. Pump is clogged. Receptacle is empty. Pump lost contact to the medium. An unsuitable medium has been used. 	<ul style="list-style-type: none"> Insert full receptacle. Insert ingo-man® plus Touchless pump into the dispenser. Remove pump and clean it. Replace receptacle. Carry out the dosage process several times. If it doesn't function, the pump is possibly clogged. Use another medium. Clean the pump thoroughly before.
5. An acoustic signal only sounds if the hand is hold below the lower sensor	<ul style="list-style-type: none"> Batteries are low. 	<ul style="list-style-type: none"> Replace batteries.
6. After insertion of the locking plate the appliance does not function as usual.	<ul style="list-style-type: none"> Appliance has not yet adjusted to the locking plate. 	<ul style="list-style-type: none"> Wait (maximum for a few minutes).
7. The appliance doses too much or not enough medium	<ul style="list-style-type: none"> Dosage adjustment is not adjusted correctly. 	<ul style="list-style-type: none"> Carry out dosage adjustment on the pump

EN 16. Disposal


16. Disposal

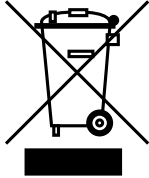Dispose of the packing material into the corresponding recycling plants.

If you definitely stop using the appliance please contact the next recycling centre or ask your specialized dealer about the applicable disposal regulations.

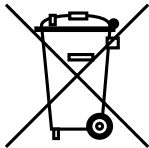16.1 Battery disposal

Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. You can dispose of old batteries at domestic collection points, public collection points in your municipality or at any location where batteries of this type are sold.


EN 17. Technical data

17. Technical data


Shroud.....	aluminium, anodized	Voltage.....	7.5 volt DC
Operating lever.....	stainless steel	Batteries.....	5 mono-cells type ANSI D
Mounting plate.....	synthetic material	Safety class system.....	IP22
Receptacle.....	synthetic material, removable 1000 ml	Dimensions in mm.....	W 92 x H 339 x D 209 mm
Pump.....	stainless steel and/or synthetics, removable	Total weight.....	1250 gram (without batteries)
Wall mounting.....	with dowels and screws	Conformity.....	RoHS compliant

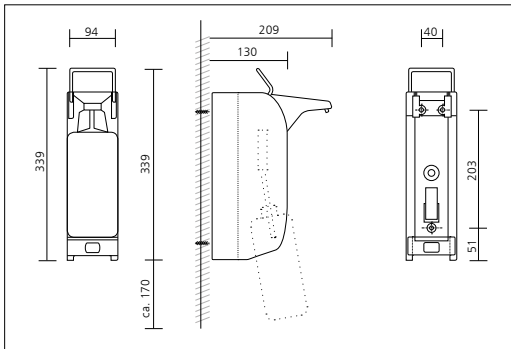Machinery directive 2006 / 42 / EC
EMC- directive 2004 / 108 / EC

IMP Touchless | Dimensions (approx.) in mm

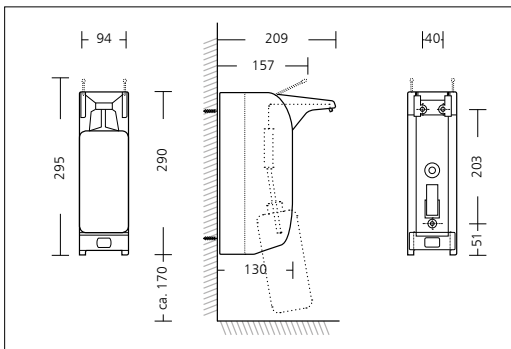
IMP Touchless XT | Dimensions (approx.) in mm

Technical modifications reserved.
Prior to the use of new filling media tests are required!

EN 18. IMP Touchless mains operated

18. IMP Touchless mains operated

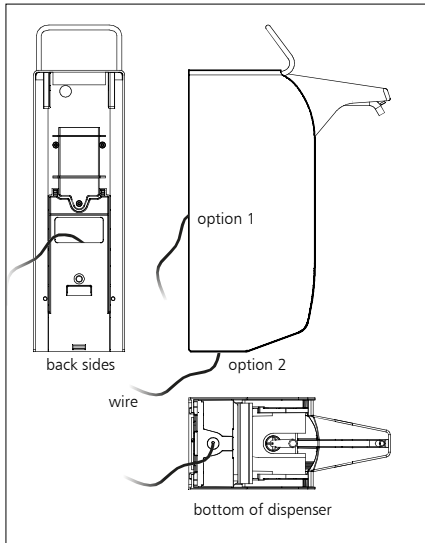The IMP Touchless also available as mains operated version.

18.1 Mains operated

Option 1

Wires enter the back of the dispensers through a hole in the backplate and a cut out in the battery cover.

Jack is connected to female plug inside the battery compartment.

Option 2

2.5 mm Jack plugs into a panel mount 2.5 mm plug on the bottom of the dispenser.

18.2 Power supply

2 m extension possible on Jack end.

Extension also possible on mains plug end.

Notizen/Notes

Notizen/Notes

OPHARDT
HYGIENE

OPHARDT HYGIENE-TECHNIK GmbH + Co. KG

Lindenau 27 · 47661 Issum · Germany

Phone +49 28 35 18-0 · Fax +49 28 35 18-77

Email customer care@ophardt.com · www.ophardt.com